Clear Anatomical Face Mask Single-Patient-Use

INDICATIONS FOR USE:

The intended use of the Clear Anatomical Face Mask is to direct gas to the upper airway.

WARNINGS:

- This mask is non-conductive and should not be used in the presence of flammable anesthetics.
- Do not sterilize, soak or wash this product. To do so may result in harmful residuals or have adverse side effects on this product.
- Do not re-use this product. To do so may result in transmission of infection or disease.

CAUTIONS:

- During use, the patient's condition must be continuously monitored.

DIRECTIONS FOR USE:

- Connect to breathing circuit or resuscitation device with appropriate fitting.
- Inspect all fittings to ensure secure connections.
- If the mask contains an inflation valve, inflate or deflate the air cushion using a needleless syringe until proper patient fit is accomplished.
- Secure by hand or use a mask strap to hold in place.

Specifications:

The MERCURY MEDICAL® Clear Anatomical Face Mask was designed to meet ISO 5356-1 requirements.

Port Size:	Adult Large	22 mm I.D.
	Adult Small	22 mm I.D.
	Child	22 mm I.D.
	Pediatric	22 mm I.D.
	Infant	15 mm O.D.
	Neonatal	15 mm O.D.

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Scanlan Group B.V.