GB INSTRUMENT FOR USE Pharma Trach**CE 0413**

Product system for tracheostomized or intubated spontaneously breathing patients. - with oxygen supply system as an addition.

Description

Single patient use Heat and Moisture Exchanger for spontaneously breathing patients.

Indications

Protection of patients from excessive loss of heat and moisture when tracheostomized or intubated and spontaneously breathing.

Contraindications.

• Device should be changed every 24 hours or more frequently if secretions accumulate. Replace immediately to a new product if the device become soiled or obstructed by secretion.

• Patient with high sputum production or haemoptysis should be evaluated closely.

Caution

• Device is intended for single patient use.

• The additional dead space for this device must be taken into consideration when used.

• In case of use of additional oxygen supply device make sure that the oxygen supply device is properly connected and oxygen tube is fixated in order to prevent drag or disturbance on the HME- device.

Recommendation

• The HME cassettes on Pharma Trach are exchangeable: In case of sensitive tracheotomy a change of cassettes instead of the entirely product could lead to a more convenient situation for the patient.

• Exchangeable HME cassettes are particular designed for patient in Home Care treatment.

Product specification:

Dead space (ml)	12
Weight(g)	4
Pressure drop(Pa) 30 l/min	70
Humidification efficiency at VT=500ml	26 mg H2O/l air
Connector: 15 F	

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Pharma Systems AB